E

This
rec
76.

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Wo
Th
pro
Th
gre
dila
Th
no
ren
inf
tre
ca

Im
●
●
rec
the
su
●
●
●
●
Do
in
●
sp
fal
●

Us
Th
th
Yo
ho
Fi
th
m

Th
In
rh
sp
W
D
sh
Th
gi
be
Th
w

English

This product is manufactured to comply with the radio interference requirements of the Council Directive of 4 November 1976 76/889/EEC; 76/890/EEC resp.

Note: These instructions for use refer to the Infraphil HP 2001. Before using the appliance, check that the typenumber (HP 2001) corresponds to the typenumber indicated on the appliance. Also check that the mains voltage in your home corresponds to the voltage indicated on the front of the bulb.

Working of the Infraphil

The "Infraphil" produces short wave infra red. An important property of this infra red is that it penetrates deeply into the skin. The pain relieving and healing action of this irradiation is thus great. The skin is heated in such a way that the blood vessels are dilated and the flow of blood is stimulated.

The blood brings with it the substances for rapidly rebuilding and nourishing the body tissues and waste products are quickly removed. The pores in the skin will also be opened under the influence of the heat. These properties make the "Infraphil" extremely suitable for therapeutic as well as for cosmetic applications.

Important

- For therapeutic applications, first consult your doctor.
 - Pain-killing preparations reduce the sensitivity to heat. An infra red treatment (for therapeutic as well as cosmetic purposes) is therefore not recommended when using such preparations. Consult your doctor.
 - Close your eyes when treating the face.
 - Avoid sudden cooling immediately after treatment.
 - Do not subject the lamp to heavy shocks.
 - Avoid water from coming into contact with the "Infraphil".
- Do not use the appliance near a swimming pool or wash stand nor in a bathroom or shower room.
- Do not place inflammable objects in the beam of light: combs or spectacle frames made from certain plastics may be damaged and fabrics may get scorched.
 - Let the lamp cool down for some minutes before storing it.

Use of the "infraphil"

The "Infraphil" HP 2001 is provided with a flexible arm (fig. 1). Slide this arm into the wall holder or the table clamp (fig. 2 and 3). You can use the screws and rawplugs supplied for fixing the wall holder.

Fix the wall holder tightening the winged nut. The flexible arm and the wall holder or table clamp enable you to position the lamp in many different positions. Fig. 4 and 5 show you some possibilities.

Therapeutic applications

Infra-red light can promote healing and provide relief from rheumatic pains, colds, muscular pain, lumbago, bruises and sprains.

We recommend that you first consult your doctor.

Distance: The distance of the appliance from the skin to be heated should be such that the heat feels comfortable (± 30 cm).

Treatment time: Generally a treatment of 15 minutes twice a day gives a better result than 30 minutes once a day. The treatment can be continued indefinitely.

The "Infraphil" can also be used for the treatment of sick or wounded domestic animals. Consult your vet.

Cosmetic applications

The "Infraphil" stimulates blood circulation, thereby helping to keep the skin smooth and clean and free from wrinkles, blackheads and clogged pores. Cosmetic preparations such as cleansing creams are more readily absorbed, and therefore more effective, when the "Infraphil" is used before and after applying the creams.

Distance: the distance of the appliance from the skin to be heated should be such that the heat feels comfortable (± 30 cm).

Cleaning and maintenance

The "Infraphil" can be cleaned with a cloth dampened with distilled water or alcohol, if so required. The lamp must be allowed to cool, and disconnected from the wall socket, before cleaning.

The bulb must only be replaced with an identical "Infraphil" bulb with the same end cap (E27).

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Guarantee Certificate

Philips warrant this appliance for a period of **one year**.

The guarantee is given against defects in material and workmanship and comprises piece parts and labour. Any costs of transport are on account of buyer.

Apply to your Philips dealer or the Philips organization in your country in case of a defect producing the completed guarantee certificate (see the outside of the box).

The guarantee is not valid if the appliance has been misused or if work has been carried out by unqualified persons.

For guarantee conditions on products bought in U.K. see separate purchase card.

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