Deutsch

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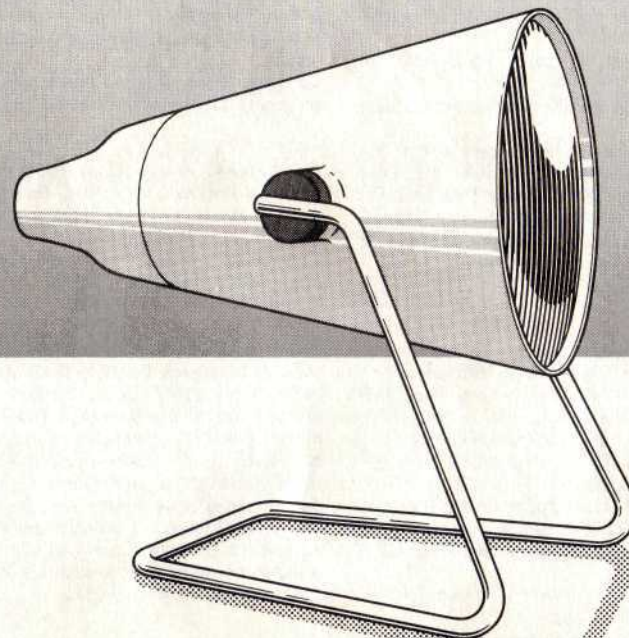
PHILIPS

English

Check whether the type number stated on the instructions for use is the same as the type number stated on the typeplate of the appliance.

This product is manufactured to comply with the radio interference requirements of the Council Directive 82/499/EEC; 82/500/EEC resp.

Before using the appliance ensure that the voltage indicated on the front of the bulb corresponds with the mains voltage in your home.

Important

- Pain-killing preparations reduce the sensitivity to heat. Infra-red treatment when using such preparations is therefore not recommended. **If in doubt consult your doctor.**
- Close your eyes when irradiating.
- Avoid sudden cooling of treated areas immediately after treatment.
- Always place the lamp on a horizontal surface. Do not suspend it from the wall or ceiling.
- Never spill water on the appliance.
- Do not use the lamp in bathrooms or in showers.
- Do not place inflammable objects in the beam of light: combs or spectacle frames made from certain plastics may be damaged and fabrics may get scorched.
- Do not subject the lamp to heavy shocks.

Working of the "Infraphil"

The "Infraphil" produces infra-red light. This light penetrates deep into the skin. The skin is therefore heated, so that the blood vessels are dilated and the flow of blood is stimulated. The blood brings with it the substances for rebuilding and nourishing the body tissues more rapidly whereas decomposition products are more quickly removed. The pores in the skin will also be further opened under influence of the heat. These properties make the "Infraphil" extremely suitable for therapeutic as well as for cosmetic applications.

Therapeutic applications

Infra-red light can promote the healing in some cases and/or relieve the pain with rheumatic affections, colds, muscular pain, lumbago, bruises and sprains. We recommend that you consult your doctor first.

Operating distance

The lamp should be placed at a distance from the skin to be heated such that the heat feels comfortable. Normally ± 30 cm (12") is ideal.

Treatment time

Generally a treatment of 15 minutes twice a day gives a better result than 30 minutes once a day. The treatment can be continued indefinitely.

Cosmetic applications

After all make-up has been removed, the pores in the skin will be opened under influence of infra-red light by which an effective cleansing of the skin is possible.

Creams or other skin care agents are better absorbed under influence of infra-red light, and are therefore most effective.

The "Infraphil" will also help to keep your skin healthy and clean, because of its stimulating effect on the blood circulation.

Operating distance: The lamp should be placed at a distance from the skin to be treated such that the heat feels comfortable. Normally ± 30 cm (12") is ideal.

Cleaning and maintenance

The "Infraphil" can be cleaned with a cloth dampened with distilled water or alcohol, if so required. Before cleaning, however, the lamp must be allowed to cool, and disconnected from the wall socket.

The bulb must be replaced by an identical "Infraphil" bulb.

Guarantee

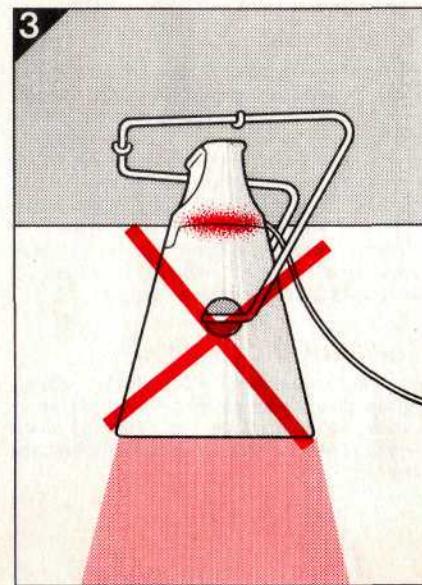
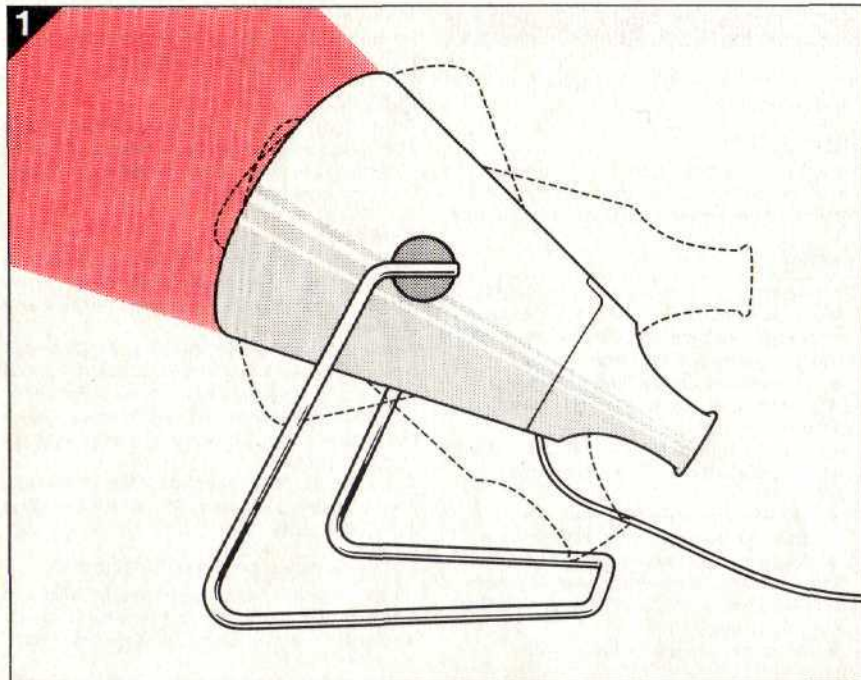
Philips warrant this appliance for a period of **one year**.

The guarantee is given against defects in material and workmanship and comprises piece parts and labour. Any costs of transport are on account of buyer.

Apply to your Philips dealer or the Philips organization in your country in case of a defect producing the completed guarantee certificate (see the outside of the box).

The guarantee is not valid if the appliance has been misused or if work has been carried out by unqualified persons.

For guarantee conditions on products bought in U.K. see separate purchase card.

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Wiring instructions - for U.K. only

IMPORTANT

The wires in this mains lead are coloured in accordance with the following code: -

**BLUE - NEUTRAL
BROWN - LIVE**

As the colours of the wires in the mains lead of this appliance may not correspond with the markings identifying the terminals in your plug, proceed as follows: -

The wire which is coloured BLUE must be connected to the terminal which is marked with the letter 'N' or coloured BLACK.

The wire which is coloured BROWN must be connected to the terminal which is marked with the letter 'L' or coloured RED.

DO NOT CONNECT either of these wires to the earth terminal in the plug which is marked with the letter 'E' or by the earth symbol \perp or coloured GREEN, or GREEN AND YELLOW.

Note If the terminals of the plug are unmarked or if you are in any doubt consult a qualified electrician.

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