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## English (see also page 22)

It is well known that in many cases heat can alleviate pain and speed up the process of recovery. This heat is administered in a very effective and practical way by means of Philips Infraphil. The Infraphil warms the skin and the underlying tissues in such a way that the blood-vessels are dilated and the flow of blood is stimulated. The blood brings with it the substances for rebuilding and nourishing the body tissues, thus healing is promoted and pain soothed away.

**First check whether the lamp matches the mains voltage in your house.**

**The voltage is indicated on the front of the bulb.**

**Distance from the skin** — The distance should be such that the warmth is comfortable without any sensation of burning. In general 12-20 inches (30-50 cm).

**Irradiation time** — There is no limit to the number of treatments. A treatment of 15 minutes twice a day usually gives the best results.

**Note** — Certain pain-killing preparations reduce the body's sensitivity to heat. Treatment with the Infraphil should not be given where these are being taken.

### Important

1. Inflammable substances should be kept away from the rays of the lamp: combs or spectacle frames, etc. might catch fire, fabrics might get scorched.
2. Keep the eyes closed when irradiating the face.
3. Avoid sudden cooling immediately after treatment (especially the head).
4. Do not subject the lamp to heavy shocks, do not put it down too abruptly nor jolt the lamp, particularly while it is still hot.
5. After switching off the lamp, allow it to cool off for some minutes before storage.

### This is where your Infraphil helps

**Rheumatic pains, aching muscles and lumbago** — Continue the treatment daily until the pain has completely disappeared. In the case of rheumatism of the joints first consult your doctor.

**Colds** — Colds can be suppressed in the early stages. If the cold has already developed, the treatment will considerably relieve the heavy feeling. Use the Infraphil for real colds only. In the case of maxillary sinusitis or frontal sinusitis first consult your doctor.

**Bruises and sprains** — Start the treatment about 12 hours after the accident and continue the treatment until all pain and stiffness have disappeared.

**Inflamed skin, boils, etc.** — Regular use of the Infraphil, combined with medical treatment, promotes rapid healing. If inflammation is extensive or recurrent always consult your doctor.

**Wounds** — Wounds will heal more quickly when irradiated with the Infraphil while the chance of infection is limited. In the case of inflamed wounds first consult your doctor.

**Insect bites** — Itching, pain and tightness of the skin can be dispelled by using the Infraphil.

**Pain after extraction of teeth** — Pain after extraction of teeth can be prevented or relieved through an Infraphil treatment.

**Chilblains, numb fingers** — A chapped skin, chilblains, numb fingers, cold feet, etc. are due to a reduced blood circulation. Treatment with the Infraphil improves the blood circulation. It is important, however, that this treatment is started in time.

### Beauty treatment

Treatment with Infraphil stimulates the blood circulation; this helps to keep the skin smooth and clean (the formation of wrinkles, blackheads and clogged pores is checked). Furthermore, cosmetic preparations are more readily absorbed when the Infraphil is used before and after applying the creams. The Infraphil is suitable for both a dry and greasy skin.

**Loss of hair** — Treatment with the Infraphil, if necessary in combination with massage of the scalp, may provide a speedy cure for **temporary** hair loss.

**Massage** — The effect of massage is considerably increased by prior or simultaneous treatment with the Infraphil, so that subcutaneous haemorrhage and accumulations of fat and fluid etc. disappear more rapidly.

### The Infraphil in sport

If the Infraphil is applied before a sporting event, the supply of blood is increased, so that the sportman's condition is improved. During the intervals use the Infraphil, if necessary, combined with massage, to dispel tiredness, muscular cramp and stiffness. Minor injuries such as sprains, bruises, etc. are treated effectively.

### Treatment of animals

The healing warmth of the Infraphil can also be used for sick or wounded domestic animals, e.g. for healing bruises, strains, muscular pain (of racehorses and greyhounds), eczema, bald patches, inflammation of the skin and nipples, swollen hocks and badly healing wounds. Also after the trimming of dogs.

### When to consult the doctor?

When the real symptoms of illness appear or when apparently trivial symptoms persist, always consult your doctor. In all those cases where warmth can promote a speedy recovery, he will undoubtedly prescribe or approve the use of the Infraphil.

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### **Working life of the lamp**

Prior to leaving the factory, the Infraphil lamp was tested and found to be working properly. With normal use, as described in the directions for use, the performance of the lamp remains constant during its entire life (about 300 burning hours).

### **Replacement of the lamp**

1. Press the bulb inwards and turn  $\frac{1}{8}$  turn anti-clockwise. Remove bulb.
2. New bulb fits only in one position. Press new bulb inwards and turn  $\frac{1}{8}$  turn clockwise.

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## English

### How to assemble the stand of the "Varia de Luxe" Infraphil®

1. Pull the three parts of the tube upwards as far as they will go. Turn the screw to the right to properly lock the two upper parts of the tube. Turn the bottom part of the tube about  $\frac{1}{3}$  turn to the right so that the blocking bolt on it engages in the recess in the base.
2. To position the base proceed as follows :
  - a) Fold out the three legs of the base
  - b) In each foot there is an extension bar which must be hinged out.  
This is absolutely necessary to ensure firm standing.
3. Place the holder for the appliance on top of the stand properly fitting it and tightening the screw.
4. Push the stand of the Infraphil lamp itself backwards through the rail of its holder towards the clamp until the stand of the appliance fits in the groove of this lamp.
5. Make sure that the stand with the Infraphil is firmly standing on a flat and level surface.
6. If you wish to remove the Infraphil from the stand, the appliance should first be allowed to cool off. Then push the clamp downwards at the rear and pull the appliance forwards from the rail.

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