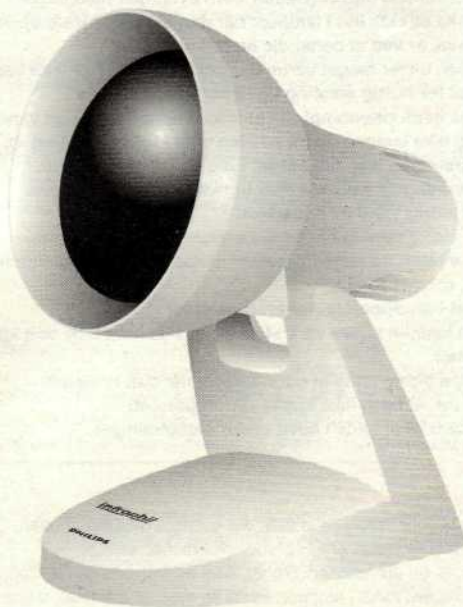
### Belangrijk

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## English

### Important

- Check if the voltage indicated on the front of the bulb corresponds to the mains voltage in your home before you use the appliance for the first time.
- Painkillers reduce the sensitivity to heat. Therefore, you are advised not to take infrared treatments when you are using such medicines (consult your doctor).
- Do not look into the lamp when it is on! Keep your eyes closed, e.g. when you are treating a cold.
- The lamp will get quite hot when it is burning. Do not touch it!
- Avoid too intense cooling immediately after the treatment.
- Avoid overheating! Always use the appliance on a horizontal surface. Do not suspend it from the wall or ceiling (fig. 1 and 2).
- Do not cover the appliance. Never put e.g. a cloth or a piece of clothing on top of it.
- Never spill water on the appliance.
- Do not use the appliance in the bathroom or in the shower.
- Keep the beam of radiation at a safe distance of at least 45 cm from inflammable objects and walls.
- Do not subject the appliance to heavy shocks.
- Allow the appliance to cool down for a few minutes before you put it away.
- Do not leave the appliance unattended when it is switched on.
- Prevent children from playing with the appliance.
- Take care not to fall asleep during a treatment.

### The effect of the Infraphil

The Infraphil produces infrared radiation which penetrates deeply into the skin. The radiation heats the skin, dilating the blood vessels and thus stimulating the blood circulation.

The blood transports the substances necessary for rebuilding and nourishing the body's tissues more rapidly, while at the same time waste substances are removed more quickly.

The heat will also cause the pores in the skin to open up.

These properties make the Infraphil extremely suitable for therapeutic as well as cosmetic application.

### Therapeutic applications

Infrared radiation can be used to treat muscular pains (fig. 3 and 4) and (common) colds (fig. 5).

The appliance may not be used if you have a swelling or an inflammation. In these cases, warmth could aggravate the complaints. If in doubt, consult your doctor.

If there should be no improvement after 6-8 treatments, stop using the appliance and consult your doctor.

### Distance and beam adjustment

During treatment of muscular pains, the distance between the lamp and the area to be treated must be such that the heat feels comfortable on the skin. This distance will usually be approx. 20 cm (8").

During treatment of colds, the distance between the face and the appliance must be approx. 35 cm (14").

Fig. 6 shows how you can aim the beam of infrared light in the desired direction.

### Duration of treatment

The duration of the treatment depends on the person to be treated and the type of application. Practice has shown that a few short treatments (e.g. 2 x 15 minutes) a day will produce a better result than one single longer treatment (e.g. 1 x 30 minutes).

### Cosmetic applications

After you have removed all make-up, the pores in the skin will open up as a result of the infrared radiation, allowing better cleaning of the skin.

Creams or other skin care products are absorbed better and more quickly by the skin as a result of infrared radiation, which increases their effectiveness.

The Infraphil will also help keep your skin healthy and clean because of its stimulating effect on the blood circulation.

### Cord storage: fig. 7.

### Cleaning and maintenance

The appliance can be cleaned with a cloth moistened with some distilled water or alcohol.

Allow the lamp to cool down and unplug the appliance before you start cleaning it. The bulb must be replaced by an identical Infraphil bulb (Philips R95, 100W).

### Replacing the mains cord

If the mains cord of this appliance is damaged, it may only be replaced by Philips or an authorised Philips service centre, as repair requires special tools and/or parts.

This appliance meets the requirements of MDD 93/42/EEC for medical appliances.

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