www.infraphil.nl www.infraphil.nl www.infraphil.nl
il.nl www.infraphil.nl www.infraphil.nl
infraphil.nl www.infraphil.nl www.infraphil.nl
il www.infraphil.nl www.infraphil.nl www.infraphil.nl
raphil.nl www.infraphil.nl www.infraphil.nl
w.infraphil.nl www.infraphil.nl www.infraphil.nl
il.nl www.infraphil.nl www.infraphil.nl
infraphil.nl www.infraphil.nl www.infraphil.nl
il www.infraphil.nl www.infraphil.nl www.infraphil.nl
raphil.nl www.infraphil.nl www.infraphil.nl
w.infraphil.nl www.infraphil.nl www.infraphil.nl
il.nl www.infraphil.nl www.infraphil.nl
infraphil.nl www.infraphil.nl www.infraphil.nl
il www.infraphil.nl www.infraphil.nl www.infraphil.nl
raphil.nl www.infraphil.nl www.infraphil.nl
w.infraphil.nl www.infraphil.nl www.infraphil.nl
il.nl www.infraphil.nl www.infraphil.nl
infraphil.nl www.infraphil.nl www.infraphil.nl
il www.infraphil.nl www.infraphil.nl www.infraphil.nl
raphil.nl www.infraphil.nl www.infraphil.nl
w.infraphil.nl www.infraphil.nl www.infraphil.nl
il.nl www.infraphil.nl www.infraphil.nl
infraphil.nl www.infraphil.nl www.infraphil.nl
il www.infraphil.nl www.infraphil.nl www.infraphil.nl

KL 7500

Binkhuysen
1963

Gebrauchsanweisung
Mode d'emploi
Gebruiksaanwijzing
Directions for use
Instrucciones de manejo

PHILIPS

infraphil

PHILIPS

infraphil

HEILWÄRME IM HEIM
CHALEUR CURATIVE
GENEZENDE WARMTE
HEALING WARMTH
EL CALOR COMO MEDIO CURATIVO

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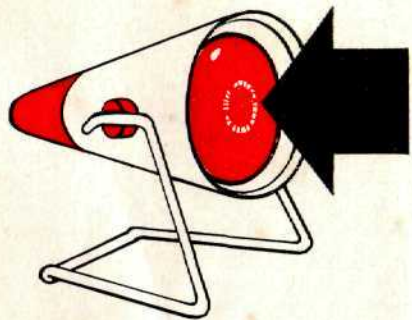
It is well known that in many cases heat can alleviate pain and speed up the process of recovery. This heat is administered in a very effective and practical way by means of Philips Infraphil. The Infraphil warms the skin and the underlying tissues in such a way that the blood-vessels are dilated and the flow of blood is stimulated. The blood brings with it the substances for rebuilding and nourishing the body tissues, thus healing is promoted and pain soothed away.

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
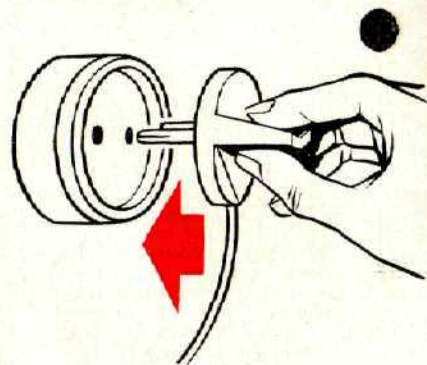
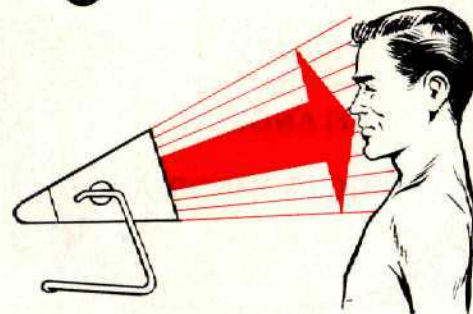
Distance from the skin — The distance should be such that the warmth is comfortable without any sensation of burning. In general 12—20 inches (30—50 cm.).

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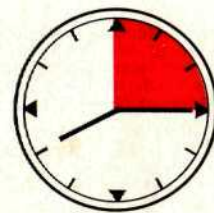Irradiation time — There is no limit to the number of treatments. A treatment of 15 minutes twice a day usually gives the best results.

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Note — After having sedatives, drugs or local anaesthesia, and also in the case of certain illnesses the sensitivity of the skin is diminished. In these cases the distance to the skin should be at least 20 inches (50 cm.).

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1. Inflammable substances should be kept away from the rays of the lamp, celluloid combs or spectacle frames, etc. might catch fire, fabrics might get scorched.
2. Keep the eyes closed when irradiating the face.
3. Avoid sudden cooling immediately after treatment (especially the head).
4. Do not subject the lamp to heavy shocks, do not put it down too abruptly or jolt the lamp, particularly while it is still hot.

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Rheumatic pains, aching muscles and lumbago — Continue the treatment daily until the pain has completely disappeared. In the case of rheumatism of the joints first consult your doctor.

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Colds — Colds can be suppressed in the early stages. If the cold has already developed, the treatment will considerably relieve the heavy feeling. Use the *Infraphil* for real colds only. In the case of maxillary sinusitis or frontal sinusitis first consult your doctor.

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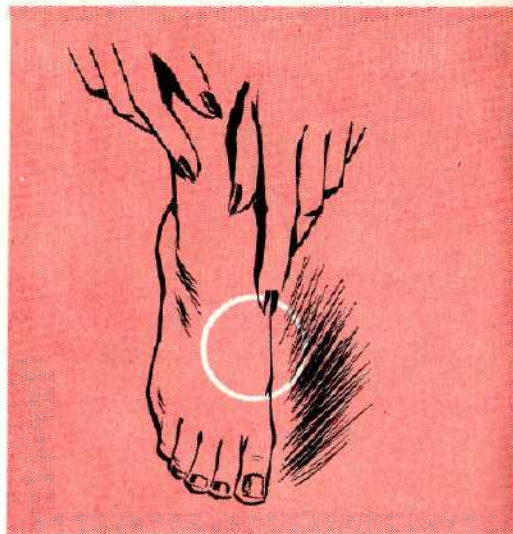
Bruises and sprains — Start the treatment about 12 hours after the accident and continue the treatment until all pain and stiffness have disappeared.

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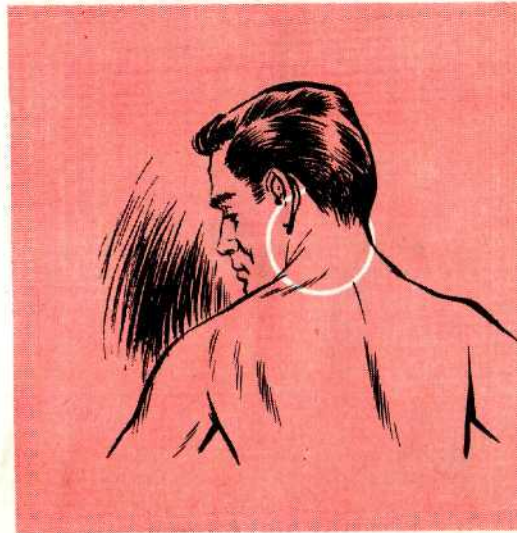
Inflamed skin, boils, etc. — Regular use of the Infraphil, combined with medical treatment, promotes rapid healing. If inflammation is extensive or recurrent always consult your doctor.

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Wounds — Wounds will heal more quickly when irradiated with Infraphil while the chance of infection is limited. In the case of inflamed wounds first consult your doctor.

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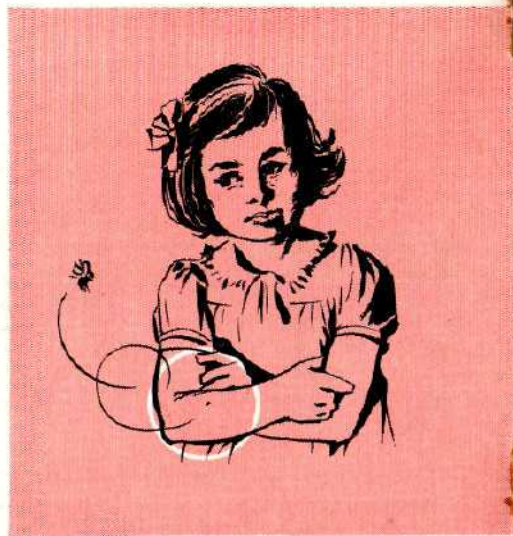
Pain after extraction of teeth — Pain after extraction of teeth can be prevented or relieved through an Infraphil treatment. (See note of page 6.)

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Insect bites — Itching, pain and tightness of the skin can be dispelled by using the Infraphil.

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When to consult the doctor?

When the real symptoms of illness appear or when apparently trivial symptoms persist, always consult your doctor. In all those cases where warmth can promote a speedy recovery, he will undoubtedly prescribe or approve the use of Infraphil.

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Treatment with Infraphil stimulates the blood circulation; this helps to keep the skin smooth and clean (the formation of wrinkles, blackheads and clogged pores is checked). Furthermore, cosmetic preparations are more readily absorbed when the Infraphil is used before and after applying the creams. The Infraphil is suitable for both a dry and greasy skin.

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Loss of hair — Treatment with the Infraphil, if necessary in combination with massage of the scalp, may provide a speedy cure for temporary hair loss.

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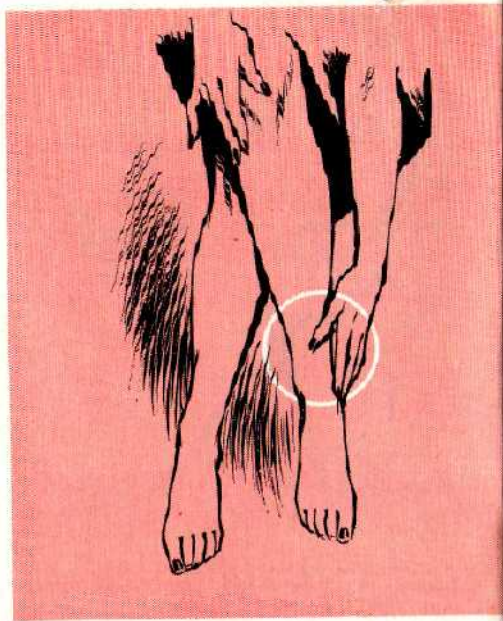
Massage — The effect of massage is considerably increased by prior or simultaneous treatment with the Infraphil, so that subcutaneous haemorrhage and accumulations of fat and fluid etc. disappear more rapidly.

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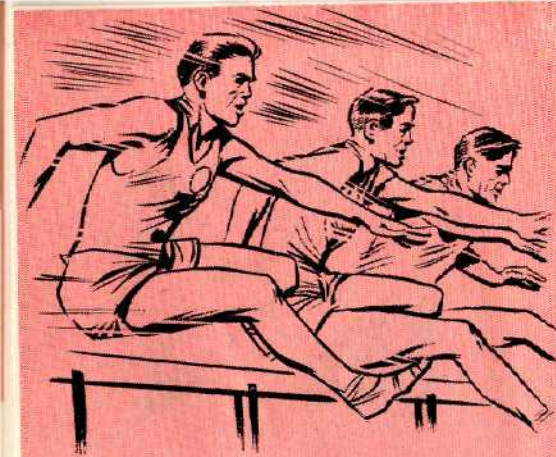
Chilblains, numb fingers — A chapped skin, chilblains, numb fingers, cold feet, etc. are due to a reduced circulation of blood. Treatment with Infraphil improves the circulation of blood. It is impor- tant, however, that this treatment is started in time.

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If the Infraphil is applied before a sporting event, the supply of blood is increased, so that the sportman's condition is improved. During the intervals use the Infraphil, if necessary, combined with massage, to dispel tiredness, muscular cramp and stiffness.

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Minor injuries such as sprains, bruises, etc. are treated effectively.

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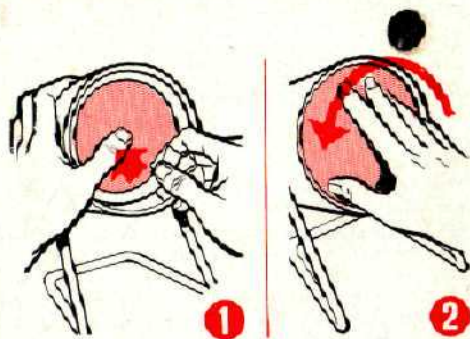
The **healing warmth of the Infraphil** can also be used for **sick** or **wounded** domestic animals, e.g. for **healing bruises**, **strains**, **muscular pain** (of **racehorses** and **greyhounds**), **eczema**, **bald patches**, **inflammation of the skin** and **nipples**, **swollen hocks** and **badly healing wounds**. Also after the **trimming of dogs**.

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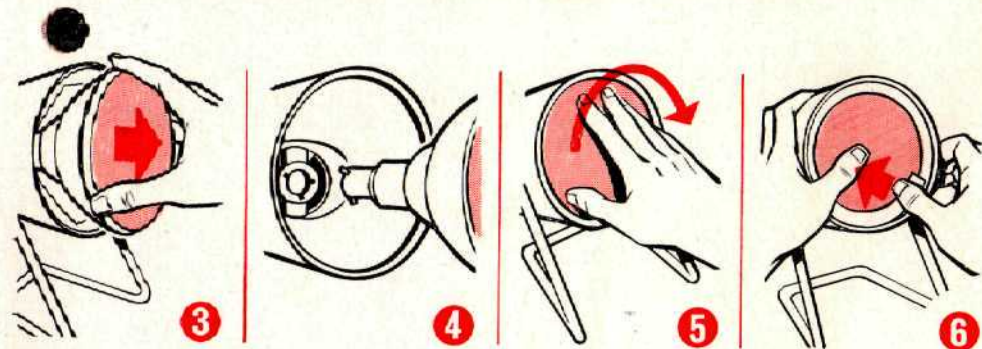
Replacement of the lamp

1. Remove the clamping ring. Press the bulb slightly inwards.
2. Press the bulb inwards and turn $\frac{1}{8}$ turn anti-clockwise.
3. Remove bulb.
4. New bulb fits only in one position.
5. Press new bulb inwards and turn $\frac{1}{8}$ turn clockwise.
6. Refit clamping ring.

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PHILIPS

infraphil

Apply to your dealer or to one of our Service work-shops if there is even the slightest deviation in the proper working of your Infraphil.

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GUARANTEE

The Infraphil lamp is guaranteed for 1 year. The performance of the Infraphil remains constant during its entire life (about 300 burning hours).

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