
PHILIPS

PR3110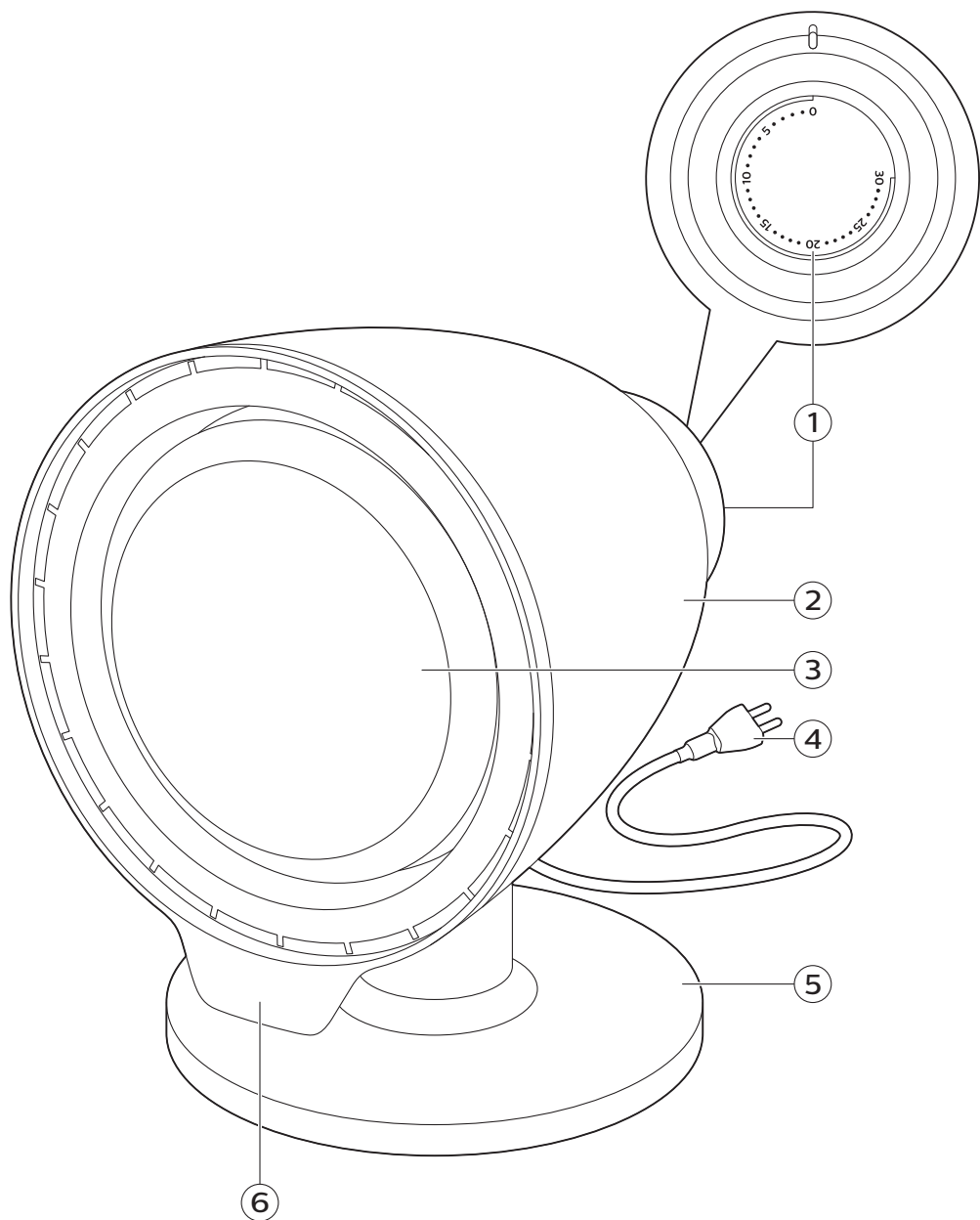

English 6
Dansk 16
Deutsch 25
Español 35
Français 45
Italiano 55
Nederlands 65
Norsk 75
Português 84
Suomi 94
Svenska 103
Ελληνικά 112

Contents

Introduction	6
Intended use	7
Important safety information	7
Contraindications	7
Warning	8
Caution	9
General	9
EMC / EMF information	10
General description	11
Preparing for use	11
Using the device	11
Duration of treatment	12
Cleaning	12
Storage	12
Replacement	12
Warranty and support	13
Recycling	13
Troubleshooting	13
Specifications	14

Introduction

Congratulations on your purchase and welcome to Philips! To fully benefit from the support that Philips offers, register your product at www.philips.com/welcome.

With the Philips InfraCare (150W), you can comfortably relieve muscle and joint pain at home. Just like the sun, the device emits infrared light.

The Philips Infrared lamp emits infrared light that is required to achieve the intended therapeutic effect. The Philips InfraCare (150W) can also be used for muscle relaxation or for preparation for a massage.

Effects of an infrared lamp

The InfraCare provides infrared light, which penetrates deeply into the skin and warms the tissues. The warmth modulates the neuronal signals to the brain and thus reduces pain. Warmth also stimulates the local blood circulation and dilates the blood vessels, which accelerates the transport of substances necessary to rebuild and nourish the body's tissues. Furthermore, infrared warmth generated by infrared light speeds up the metabolic process and the removal of metabolic waste products from the body. Warmth reduces stiffness and makes joints more flexible because it makes tissue more pliable. The benefits described can help to alleviate pain due to muscle and joint problems.

Note: Read the 'Warning' section in chapter 'Important safety information' and follow the instructions before you use the device.

Intended use

The infrared lamp is a local warmth therapy device. Possible therapeutic applications are:

- Treatment of occasional or chronic muscle pain, stiff muscles and stiff joints.
- Treatment of lower-back pain.
- Treatment of pain due to rheumatic disorders (during the non-inflammatory phase)

This device is intended for indoor use by adults who experience acute and chronic muscle or joint pain.

Other applications

You can also use the infrared lamp for the following purposes:

- Muscle relaxation.
- Preparation for a massage.

Important safety information

Read this important information carefully before you use the device and save it for future reference.

Contraindications

- Do not use the device without consulting your doctor first if you suffer from severe diseases such as heart disease, acute inflammatory diseases, thrombosis, blood coagulation disorders, adrenal suppression, systemic lupus erythematosus or malignant diseases.
- Do not use the device if you are oversensitive to infrared light or if you have oedema. If in doubt, consult your doctor.
- Do not use the device if you are insensitive to infrared radiation and heat due to a disease (e.g. advanced diabetes) or the use of painkillers. If in doubt, consult your doctor.
- Do not use the device if you suffer from conditions associated with nerve demyelination such as carpal tunnel syndrome, multiple sclerosis and ulnar neuropathy. If in doubt, consult your doctor.
- Do not use the device if you have an implanted device, particularly not if it is located in the vicinity of the treatment area. If in doubt, consult your doctor.
- If you are pregnant, do not use the device to treat areas close to your abdomen, to avoid overheating of this area. If in doubt, consult your doctor.
- Do not use the device if you have a swelling or an inflammation, as heat could aggravate the complaints. If in doubt, consult your doctor.
- Do not use the device on damaged or irritated skin or on wounds. If in doubt, consult your doctor.
- Do not use the device to treat areas on which you have recently applied topical counterirritants (creams or ointments containing e.g. menthol), as this can cause a mild inflammatory reaction.
- Do not use the device if you suffer from poor thermal regulation or impaired circulation. If in doubt, consult your doctor.

Warning

- Water and electricity are a dangerous combination. Do not use this device in wet surroundings (e.g. in the bathroom or near a shower or swimming pool).
- Do not use the device when the glass is damaged or missing, or if the coating of the glass is damaged.
- Do not touch the glass or front part of the device when it is switched on. The surface is very hot and may cause burns on your skin (see 'Caution').
- Never insert any object into the device. This may cause malfunction of the device or even cause an electrical shock.
- Do not let water run into the device or spill water onto the device.
- Check if the voltage indicated on the device corresponds to the local mains voltage before you connect the device.
- If the mains cord is damaged, you must have it replaced by Philips, a service center authorized by Philips or similarly qualified persons in order to avoid a hazard.
- Do not leave the device unattended when it is switched on.
- This device is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the device by a person responsible for their safety.
- This device is not intended for use by children. Children should be supervised to ensure that they do not play with the device.
- Always protect the device against unauthorized use by storing the device according to the storage instructions (see 'Storage').
- Keep the appliance and its cord out of the reach of children and pets to avoid the risk of burns and other injuries due to contact with the device or fall of the device. Cords present a potential strangulation hazard. Keep cords out of the reach of children and pets.
- This device is intended for the treatment of muscle and joint pain and muscle and joint stiffness, not for the treatment of abdominal, facial and other sorts of pain.
- Prevent overheating of the skin, the warmth should be comfortable throughout the treatment.
- To prevent eye damage, avoid eye exposure to the light source within 1 meter from the device. Do not look in the light source of the lamp, this may cause eye damage.
- If there is no improvement after 6-8 treatments, stop using the device and consult your doctor.
- Do not fall asleep during treatment.
- Do not modify this equipment.
- Do not place flammable and/or heat sensitive objects (such as electronic devices) in the vicinity of the device.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally."
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Caution

- If the wall socket used to power the device has poor connections, the plug of the device becomes hot. Make sure you plug the device into a properly connected wall socket.
- Do not subject the device to heavy shocks.
- Always unplug the device after use and in case of a power failure.
- Place the device on a stable, level surface (Fig. 2). Keep at least 15 cm of free space around the device to prevent overheating (Fig. 3).
- Do not cover the device with, for example, a cloth or a piece of clothing when it is switched on.
- Let the device cool down for approx. 15 minutes before you touch, clean or store it.
- If you have just been swimming or have just taken a shower, make sure you dry your skin properly before you use this device.
- Never use the device when light bulb is damaged, broken or missing.

General

The warning signs and symbols are essential to ensure that you use this product safely and correctly and to protect you and others from injury. Below you find the meaning of the warning signs and symbols on the label and in the user manual.

This symbol indicates that you have to read the user manual carefully before you use the device. Save the user manual for future reference.

This symbol warns you against hot surfaces.

This symbol indicates that the user should consult the user manual for important safety information such as contraindications, warnings and cautions that cannot, for a variety of reasons, be presented on the device itself.

This symbol indicates that this device is double insulated (Class II). This device meets the requirements of Medical Device Directive 93/42/EEC.

This symbol indicates the manufacturer: Philips Consumer Lifestyle B.V, Tussendiepen 4, 9206 AD Drachten, The Netherlands.

This symbol indicates that this product shall not be disposed of with normal household waste (2012/19/EU). See chapter 'Recycling'.

This symbol indicates the manufacturer's serial number to identify a specific device. The first 8 digits of the serial number indicate the production date: digits 1-4 indicate the year, digits 5-6 the month and digits 7-8 the day.

The symbol for the Green Dot ('Der Grüne Punkt' in German). The Green Dot is the license symbol of a European network of industry-funded systems for recycling the packaging materials of consumer goods.

This symbol indicates the relative humidity limits to which the device can be safely exposed: 15% to 93%.

10 English

This symbol indicates that an object is capable of being recycled – not that the object has been recycled or will be accepted in all recycling collection systems.

This symbol indicates that 'the device complies with European Medical Device Directive 93/42/EEC requirements'. 0344 refers to the notified body.

This symbol indicates the storage and transportation temperature limits to which the infrared lamp can be safely exposed: -25°C to +70°C.

IP21

This symbol indicates: protected against access to hazardous parts with a finger or objects of similar size and against vertically falling water drops.

This symbol warns you against optical radiation from the device which may cause eye irritation.

EMC / EMF information

- This Philips device complies with all applicable standards and regulations regarding exposure to electromagnetic fields.
- Use of the InfraCare device adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, the InfraCare device and the other equipment should be observed to verify that they are operating normally.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the InfraCare device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Environment

The Philips InfraCare infrared lamp is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.

Test levels

This device is compliant to 60601-1-2 Ed. 4 RF emissions Group 1 class B. RF emissions are very low and are not likely to cause interference in nearby electronic equipment.

Immunity test levels	Discharge level direct 8 kV, indirect 8 kV
Electrical fast transient burst	2 kV
Surge	1 kV line to line
Voltage dips short interruptions and voltage variations on power supply input lines IEC61000-4-11	70% 25 cycles (50 Hz) 30 cycles (60 Hz)
Power frequency magnetic field immunity test IEC61000-4-8	30 A/m

Immunity to conducted disturbances, induced by radio frequency (RF) fields according IEC61000-4-6 in Home Health Care environments

3 V RMS outside the ISM band, 6 V RMS inside the ISM and amateur radio bands (150 KHz– 80 MHz)

Radiated RF in Home Health Care environments for 80 MHz – 2.7 GHz according IEC61000-4-3 10 V/m

General description

- 1 Timer knob
- 2 Lamp housing
- 3 Philips Infrared incandescent Lamp
- 4 Mains cord
- 5 Pedestal
- 6 Adjustment flap

Preparing for use

Note: Do not suspend the device from a ceiling or hang it on a wall.

- 1 Fully unwind the mains cord.
- 2 Place the device on a stable, level surface (Fig. 2).
- 3 Keep at least 15 cm of free space around the device to prevent overheating (Fig. 3).

Using the device

- 1 Sit or lie down at the right distance from the device.
 - Remove clothing and jewelry from the area to be treated.
 - The infrared light needs to be used on bare skin.
 - At a distance of 30 cm to the body, the device heats an circular area of about 20 cm. At a larger distance, the heated area is larger, but the infrared light and the heat are less intense. At a smaller distance, the heated area is smaller, but the infrared light and the heat are more intense.

Warning: Make sure the distance between your body and the device is large enough to prevent accidental contact with the light bulb.

- 2 Insert the plug into a wall socket.
- 3 Position the device properly. Below you find some examples.
 - To treat a backache, place the device on a table at a distance of approx. 30 cm (Fig. 5).
 - To treat a sore knee, place the device on a low table. Sit down on a chair next to the table at a distance of approx. 30 cm (Fig. 6).
- 4 To switch on the device, turn the timer knob to the desired treatment time (Fig. 7).
 - You can choose a treatment time between 0 and 30 minutes.
- 5 Aim the beam of infrared light at the body part to be treated, by tilting the lamp housing backwards (Fig. 4).
 - Choose a distance at which the heat feels comfortable on your skin. Increase the distance if the heat becomes too intense.
- 6 The device will turn off automatically when the desired treatment time has elapsed.
 - If you want to turn off the device before the desired treatment time has elapsed, turn the timer knob to 0 minutes.

Note: If you disconnect the device from the wall socket while the timer is activated, the timer continues to count down and make a ticking sound.

Warning: To prevent eye damage, do not look into the light of the lamp when it is on. You do not need to wear goggles as the InfraCare (150W) has a special light bulb that filters out UV light.

Duration of treatment

The duration of the treatment depends on the type of treatment and the person to be treated. Treatment of a single muscle or joint should last approximately 15 minutes. If necessary, the treatment can be repeated several times a day. You can repeat the series of several treatments a day on several consecutive days to get a result. If there is no improvement after 6-8 treatments, stop using the device and consult your doctor. For chronic problems, the device can be used for longer periods of time. Follow the instructions for safe use (see 'Warning').

Warning: Always consult your doctor in case of doubt.

Cleaning

Note: Never use scouring pads, abrasive cleaning agents or aggressive liquids such as petrol or acetone to clean the device.

- 1 Unplug the device and let the lamp cool down for approx. 15 minutes before you clean the device.
- 2 Clean the device and the pedestal with a moist cloth.

Warning: Do not let water run into the device or spill water onto the device.

Storage

Note: Do not let the cord come into contact with the lamp when it is hot.

- 1 Unplug the device and let the lamp cool down for approx. 15 minutes before you store it.
- 2 Roll up the mains cord and wrap the Velcro around it (Fig. 9).
- 3 Store the device in a dry place.

Replacement

Lamp

The effectiveness of the lamp does not diminish in the course of time.

Warning: Only use a Philips PAR38 IR 150W E27 230V lamp to replace the lamp of the device. Use of any other lamp may be hazardous.

Note: If the lamp stops working, contact the Consumer Care Center in your country.

If you want to replace the light bulb of the Infracare (150W), follow the steps below.

- 1 Unplug the device and let the light bulb cool down for approx. 15 min before you touch and replace it.
- 2 Remove the light bulb by gently turning the light bulb counter clockwise until the light bulb is released from the socket.
- 3 Place the new light bulb in the socket and gently turn the light bulb clockwise until it clicks into place.
- 4 Dispose of the defect light bulb as described in section 'Recycling.'

Mains cord

If the mains cord is damaged, you must have it replaced by Philips, a service center authorized by Philips or similarly qualified persons in order to avoid a hazard.

Warranty and support

If you need information or support, please visit www.philips.com/support or read the international warranty leaflet.

Recycling

- This symbol means that this product shall not be disposed of with normal household waste (2012/19/EU) (Fig. 10).
- Follow your country's rules for the separate collection of electrical and electronic products. Correct disposal helps prevent negative consequences for the environment and human health.

Troubleshooting

This chapter summarizes the most common problems you could encounter with the device. If you are unable to solve the problem with the information below, visit www.philips.com/support for a list of frequently asked questions or contact the Consumer Care Center in your country.

Problem	Possible cause	Solution
The device does not switch on.	The plug is not inserted into the wall socket properly.	Insert the plug in the wall socket properly.
	The treatment time is not set.	Select a treatment time by turning the timer knob to the preferred treatment time.
	There is a power failure.	Connect another device to check if the power supply works.
	The lamp is defective.	Contact the Philips Consumer Care Center.
	The mains cord is damaged.	If the mains cord is damaged, you must have it replaced by Philips, a service center authorized by Philips or similarly qualified persons in order to avoid a hazard.

14 English

Problem	Possible cause	Solution
The device makes a ticking sound after disconnecting the device from the wall socket.	The timer is not in the off position.	Turn the timer to the off position.

Specifications

Model	PR3110
Rated voltage (V) and rated frequency (Hz)	220-240 V ~50-60 Hz
Rated power input (W)	150 W
Medical device classification	Ila
Insulation class	Class 2
Ingress of water	IP21
IR type	A/B/C
Expected service life	5 years
Mode of operation	Continuous
Net weight	Approximately 1kg
External dimensions	212x160x215 mm (LxWxH)
Irradiance	3323352 mW/m ²
Radiance	11654169 mW/sr-1m ⁻²
Maximum variation of the output from the mean value across the treatment area	1350 W/m ²

Infrared light

Wavelength	700-3000 nm
Max output	3323 W/m ²
Tested According to	IEC 60601-2-57:2011

Operating conditions

Temperature	From +5°C to +40°C
Relative humidity	From 15% up to 93% (no condensation)
Pressure range	700 hPa to 1060 hPa

Storage conditions

Temperature	From -25°C to +70°C
-------------	---------------------

Storage conditions

Relative humidity	Up to 93% (no condensation)
Atmospheric pressure	700 hPa to 1060 hPa

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All rights reserved.